

SPIRALE®

DRUG DELIVERY SYSTEM

INSTRUCTIONS FOR USE

Unlock the Spirale® by rotating the two end connectors against each other.

WARNING: No drug delivery will occur when the MDI is depressed if the Spirale is in the locked position.



Fully expand Spirale® by pulling the end connectors in opposite directions, taking care not to twist the extending membrane.



Depress MDI canister once during the inspiratory phase of the ventilator cycle (or patient's inspiratory breath if breathing spontaneously) or more preferably, during expiratory pause, just before the start of the inspiratory phase.

Allow a minimum of 30 seconds before any subsequent MDI canister depression, subject to physicians' prescribing guidelines.

After use, immediately remove MDI canister nozzle from the actuator port; collapse Spirale® and lock.

Failure to lock the device may cause it to spontaneously expand, creating dead space volume, leading to circuit leaks or poor circuit compliance.



Contact our team for further information and support

JOHN PAXMAN
+61 (0) 434 361 558

MICHAEL SCHIRMER
+61 (0) 422 458 810

CUSTOMER SERVICE
+61 (0) 8 6162 9911